

Lacelle Diamond Daily Contact Lens

Reg. No. GB22112995018

1. Intended use

It's disposable single focus lens to be worn directly in human eye for cosmetics and vision correction.

2. Instruction for use and Precautions

2.1. Instruction for use

2.1.1. Preparation prior to use

2.1.1.1. Carefully follow the handling and directions on the label or eye care professional before lens application.

2.1.1.2. Always wash and rinse your hands thoroughly before handling your lenses and after.

2.2. Instruction for wearing

2.2.1. When wearing the lens

2.2.1.1. Cleanse the lenses with saline solution.

2.2.1.2. Place the lens on the tip of your right index finger, place the middle finger of the same hand close to your lower eyelashes and pull down the lower lid; use the index finger of the other hand to lift up the upper lid.

2.2.1.3. Slowly take the lens to the cornea and gently place it on the eye. Do not close the eyes while doing this.

2.2.1.4. When the lens is correctly in place, remove the fingers from position.

2.2.2. When removing the lens from your eyes.

2.2.2.1. Make sure the lens is on the cornea before attempting to remove it.

2.2.2.2. Look up and slowly pull down your lower lid with the middle finger and place your index finger on the lower edge of the lens. Squeeze the lens gently between the thumb and index finger and remove it.

3. Storage method after use

This lens is disposable use only. It must be disposed after use and not to be re-used.

4. Precautions/Contraindications

- 4.1. Any eye disease, injury or abnormality or allergic reactions that may be induced or exaggerated by contact lenses.
- 4.2. Must dispose daily wear contact lenses after use.
- 4.3. Confirm with an eye care professional in case of any systemic disease that may affect the eyes.
- 4.4. Use appropriate cleaning and re-wetting solutions.
- 4.5. If eyes become irritated or cause severe pain, stop the use and seek care immediately.
- 4.6. If vision becomes blurred due to dry lenses, re-wet the eyes with eye drops for contact lenses and blink to re-focus.
- 4.7. Rest the eyes and do not wear contacts in case of fatigue, lack of sleep, and reading for a long time.
- 4.8. Saline solution, cleaner, and storage solutions must be the certified products and never use regular tap water or unknown solution.
- 4.9. Do not have the lenses in contact with soap, shampoo, spray and cosmetic products. Put make up on after wearing the lenses.
- 4.10. Be careful not to scratch the surface of the lens with finger nails.
- 4.11. Never wear contact lenses previously worn by someone else.